

# Clinichem



Original



Test approved

- Uniform black/dark brown in 134 °C cycle
- Uniform black/dark grey in 121 °C cycle



**Air pocket (light spot in or near the centre of the sheet)**

- Door gasket or throughput of the probe, or ventilator or pipe joint leaks.
- Insufficient vacuum
- Service needed



**Over sterilization**

- Temperature too high in relation to the pressure, or too long exposure time or both.



**Wet steam or non condensable gases (spots/figures/circle formed figure in the center)**

- Pressure or temperature or both too low.
- Insufficient drying.
- Steam contains non condensable gases, most often carbon dioxide.